





GET THE RELIEF YOU DESERVE

AID Ear Clip Protocol

CRANIAL ELECTROTHERAPY

1. Insert plug from ear clips into the device.
2. Saturate each felt pad with 2-3 drops of solution.
3. Place ear clips on ear lobes.
4. Press the power button on.
5. Increase current intensity slowly with the "up" arrow until a slight vertigo is felt (a floaty feeling, similar to rocking on a boat), then decrease immediately until the dizziness stops. Also decrease immediately if the normal tapping sensation felt on your earlobes is uncomfortable.
6. Set the timer to 20 min if the current intensity is 2.5 or higher. Select the timer setting for 60 min if the current intensity is 2 or less.

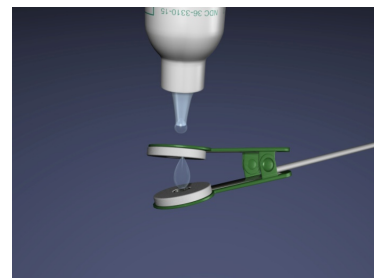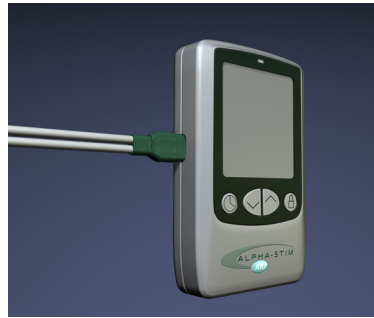

Tips

- Do not treat at uncomfortably high current levels! Too much current can cause vertigo or nausea and does not lead to better results.
- It is advised to complete a CES treatment session (20 or 60 minutes) once you start it. Do not treat for only a few minutes.
- Treat daily for 2 to 4 weeks and then every other day. Plan to treat 8 weeks before drawing conclusions about efficacy.
- Replace the felt pads as needed. Replace batteries with **lithium** AAA batteries when the battery indicator indicates no power.
- Questions? Please call us at 800-684-9343. Your Alpha-Stim® comes with free tech support.

Use Lithium Batteries Only!



Authorized Alpha-Stim® Sales & Service
Phone: (800) 684-9343
www.AlleviaHealth.com